




POLAR CARE® CUBE™



POLAR CARE® CUBE™

You deserve the advantages of motorized cold therapy. You deserve **Polar Care Cube**.

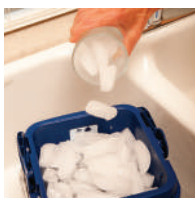

Easy setup

Just fill the unit with water to the fill line, add ice, and plug into an electrical outlet. Sit back, relax, and your Polar Care Cube will take it from there.



Easy cleanup

After use, pour out the water from the unit's fluid reservoir and remove any water remaining in the WrapOn Polar pad. Now you're all set for the next round of convenient, cold therapy pain relief.



Scan QR code for Breg's instructional video.

breg.com/PCC



Managing pain with motorized **cold therapy.**

Breg, Inc.

2885 Loker Ave. E, Carlsbad, CA 92010 USA

Toll Free Tel: 1-800-321-0607

Local: +1-760-795-5440 Fax: +1-760-795-5295

©2016 Breg, Inc. All rights reserved. AW-1.93150 Rev D 03/16
Polar Care is a registered trademark and Cube is a trademark of Breg, Inc.

POLAR CARE® CUBE™

Breg provides a suite of orthopedic solutions to improve outcomes and provide a better patient experience. With Polar Care Cube, Breg is helping patients achieve more comfortable recoveries after surgery.

“I love my Polar Care Cube! This is my second surgery, and when my knee aches or swells, I just add water and ice, and away the pain and swelling goes.”

David Ackerman

How cold therapy helps you recover

Cold therapy reduces the pain and swelling typical of orthopedic injury or surgery. Many patients with injuries like yours find the Polar Care Cube much more convenient and comfortable than gel packs or ice. Ask your doctor about the benefits of cold therapy with the Polar Care Cube from Breg.

Motorized cold therapy from Polar Care Cube

- Reduces pain and swelling
- Delivers a constant, comfortable temperature
- Is more convenient and less messy than ice or gel packs
- Polar Care Cube WrapOn Polar pads are designed to conform to injured areas, delivering cold where you need it

Ask your doctor about cold therapy risks

As with other cold treatment options such as ice or gel packs, motorized cold therapy can cause serious injury if improperly used. Please ensure you have received proper instructions from your doctor and understand the risks before using any cold therapy device.

Your **personal** cold-treatment therapist.



1. Post Operative Treatment

Based on your doctor's protocol, Polar Care Cube can be a smart investment in your recovery.

2. Physical Therapy

The Polar Care Cube helps you manage pain after physical therapy or post-workout.



3. During Recovery

Designed to get you back to regular activity, the Polar Care Cube is lightweight, compact, quiet, and easy to use.

